

Products

- = LIGASANO® white
- = LIGASANO® orange
- = LIGASANO® green









Prescription







LIGASANO® Product Family

Quick Guide



















Art.-No.	Shipping Unit
----------	---------------









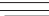
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
LIGASANO® Wundputzer® Wound Cleaning Sponges

16814-030	30 pcs.	5 x 5 x 2 cm	sterile		
36814-030	30 pcs.	5 x 5 x 2 cm	sterile		
15342-030	30 pcs.	5 x 5 x 2 cm	sterile		
36815-005	5x3 pcs. à	5 x 5 x 2 cm	sterile		







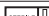

16812-020	20 pcs.	30 x 1.5 x 0.6 cm	sterile		
36812-020	30 pcs.	30 x 1.5 x 0.6 cm	sterile		
15372-020	30 pcs.	30 x 1.5 x 0.6 cm	sterile		

LIGASANO® white: Sterile Wound- and Cavity-Dressings





15341-010	10 pcs.	5 x 5 x 1 cm	sterile		
15342-010	10 pcs.	5 x 5 x 2 cm	sterile		
15346-010	10 pcs.	10 x 10 x 1 cm	sterile		
15347-010	10 pcs.	10 x 10 x 2 cm	sterile		
15350-015	15 pcs.	15 x 10 x 0,5 cm	sterile		
15351-010	10 pcs.	15 x 10 x 1 cm	sterile		
15356-010	10 pcs.	15 x 10 x 2 cm	sterile		
15352-010	10 pcs.	24 x 16 x 1 cm	sterile		
15357-005	5 pcs.	24 x 16 x 2 cm	sterile		

15370-010	10 pcs.	300 x 2.5 x 0.4 cm	sterile		
15371-007	7 pcs.	100 x 1.5 x 0.4 cm	sterile		
15372-010	10 pcs.	50 x 1.5 x 0.4 cm	sterile		
15375-010	10 pcs.	6 x 2.5 x 0.4 cm	sterile		
15385-010	10 pcs.	Ø 5 cm x 1 cm	sterile		

LIGASANO® white: Sterile und Unsterile Bandages





15381-008	8 pcs.	300 x 5 x 0.3 cm	sterile		
15383-004	4 pcs.	300 x 10 x 0.3 cm	sterile		
15571-012	12 pcs.	300 x 5 x 0.3 cm	unsterile		
15573-006	6 pcs.	300 x 10 x 0.3 cm	unsterile		



LIGASANO® white: Dressing for Fingers and Toes with Adhesive Stripe

15394-015	15 pcs.	14 x 10 x 0.5 cm T-shaped dressing for tubes and caps sterile		
15215-001	1 piece	200 x 14 x 0.5 cm unsterile rolled, can cut to fit, with adhesive stripe for ca. 40-50 tubes for toes		




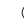


Art.-No.	Shipping Unit
----------	---------------

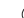LIGASANO® white: Unsterile Dressing

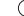


15152-010	10 pcs.	24 x 16 x 1 cm	unsterile	
15153-005	5 pcs.	24 x 16 x 2 cm	unsterile	
15158-013	13 pcs.	15 x 10 x 2 cm	unsterile	
15159-026	26 pcs.	15 x 10 x 1 cm	unsterile	

15083-025	25 pcs.	Ø 3.5 cm x 1 cm	unsterile, round	
15085-036	36 pcs.	Ø 5 cm x 1 cm	unsterile, round	









LIGASANO® white and green: Large Unsterile Dressings

12001-007	7 pcs.	59 x 49 x 2 cm	unsterile	
12002-014	14 pcs.	59 x 49 x 1 cm	unsterile	
12006-001	1 pc.	200 x100 x 2 cm	unsterile	
12007-002	2 pcs.	200 x100 x 1 cm	unsterile	
16001-007	7 pcs.	55 x 45 x 2 cm	unsterile	
16006-001	1 pc.	190 x 90 x 2 cm	unsterile	


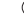



LIGASANO® white and green: Palm Dressing

15090-012	10 pcs.	29 x 12 x 1 cm	unsterile	
	2 pcs.	Ø 8 cm, 10 cm wide		
15091-010	10 pcs.	29 x 12 x 1 cm	unsterile	

LIGASANO®Roll: Dispenser Boxes

15025-001	1 pc.	200 x 2.5 x 1 cm	unsterile	
15225-001	1 pc.	300 x 2.5 x 0.6 cm	unsterile	
15050-001	1 pc.	300 x 5.0 x 0.3 cm	unsterile	
15055-001	1 pc.	200 x 5.5 x 1 cm	unsterile	
15255-001	1 pc.	300 x 5.5 x 0.6 cm	unsterile	
15110-001	1 pc.	200 x 11 x 1 cm	unsterile	
15210-001	1 pc.	300 x 11 x 0.6 cm	unsterile	
16110-001	1 pc.	200 x 11 x 1 cm	unsterile	

LIGASANO® white and green: Planter Dressing

15098-007	4 pcs.	35 x 15 x 1 cm	unsterile	
	1 pcs.	35 x 15 x 2 cm	unsterile	
	5 pcs.	35 x 15 x 1 cm	unsterile	
	1 pcs.	35 x 15 x 2 cm	unsterile	
	1 pcs.	Stocking for fixation		
15099-020	20 pcs.	35 x 15 x 1 cm	unsterile	
		Follow-up package		

LIGASANO® can be used both therapeutically and preventively. Here are some examples for prescriptions for **therapeutical use**, also apart from the direct wound treatment.

Proposals for the prescription text


wound treatment: wound cleaning germ reduction promotion of granulation promotion of epithelisation	... OP LIGASANO® due to decubital ulcer category ... (2-6) ... OP LIGASANO® due to arterial ulcus cruris ... OP LIGASANO® due to venous ulcus cruris ... OP LIGASANO® due to arterial-venous ulcus cruris ... OP LIGASANO® due to diabetes resulted foot lesions ... OP LIGASANO® for wound cleaning ... OP LIGASANO® due to scratches / abrasions ... OP LIGASANO® due to burns ... OP LIGASANO® for drainage of the rest area in case of heavy exudating wounds ... OP LIGASANO® due to traumatic wound by OP LIGASANO® for postoperative treatment ... OP LIGASANO® due to anus praeter
bedridden patients	... OP LIGASANO® due to highly disorders of blood circulation at rest area or risk regions
intertrigo prophylaxis	... OP LIGASANO® due to an adiposity qualified risk of intertrigo
intertrigo treatment	... OP LIGASANO® due to intertrigo in the ... region
fungal infections e.g. in skin crinkles	... OP LIGASANO® for an accompanied treatment of fungal infections
arterial disorders of blood circulation	... OP LIGASANO® bandages due to highly arterial disorders of blood circulation of the legs
venous afflux in the legs	... OP LIGASANO® bandages as a support of a compression therapy
maceration	... OP LIGASANO® because of skin lesions by maceration

Further therapeutical applications arise from the wide utilisation spectrum

Prescription



LIGASANO® Product Family

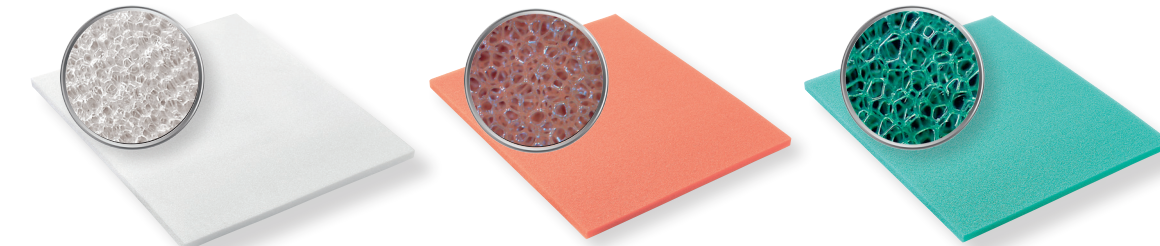

LIGASANO® white:

- Mixed-pored, relatively macropored foamed polyurethane
- Cell type 750

Properties:

- rapidly falling pressure tension



- mechanical stimulus



- action effect




LIGASANO® orange:

- Open-pored and large-pored grid with removed cell membranes
- Cell type 850

Properties:

- open-pored and large-pored structure
- moderate mechanical stimulus
- nearly completely permeable to steam

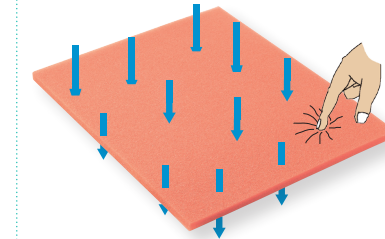


LIGASANO® green:

- Open-pored and very large-pored grid with removed cell membranes
- Cell type 1500

Properties:

- open-pored and large-pored structure
- intensive mechanical stimulus
- nearly completely permeable to steam

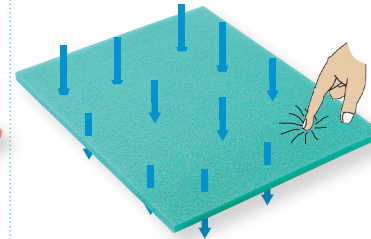

LIGASANO®

Wound Cleaning - Wound Healing - Prevention



If you like to get sent our complete product list with **all LIGASANO® product** please contact us via **info@ligamed.de**

Wound Cleaning

Wound Healing

Prevention

Sale / Delivery


Debridement is an important element of wound treatment. LIGASANO® sterile with the additional name "Wundputzer®" (wound cleaning sponge) usually remains in the wound only for a short time to clean it. A clean wound is absolutely required for wound healing and wound documentation.

Depending on type and amount of the coatings, three types of wound sponges are available, a combination can be useful. For optimal cleaning results, please use LIGASANO® Wundputzer® with **rotation technique** (rotating about 180°) or **wipe technique** (wiping in one direction only).

Where cleaning is difficult e.g. between toes, in wound pockets or in undermined wounds, the LIGASANO® Wundputzer® (wound cleaning sponges) interdigital will help you. They have the same properties as our regular wound cleaning sponges and are only adapted in form for special applications.


LIGASANO® Wundputzer® intensive

Removes very persistent, viscous, firm and/or necrotic coatings, may absorb a lot of detritus.

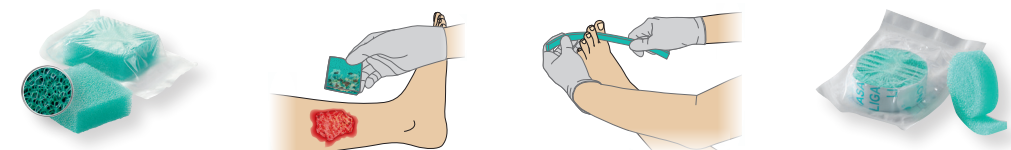


LIGASANO® Wundputzer® interdigital intensive

Removes very persistent, viscous, firm and/or necrotic coatings, may absorb a lot of detritus.

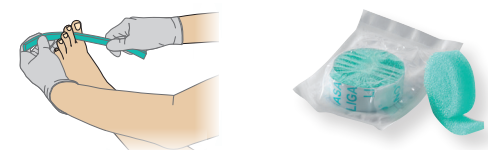


LIGASANO® Wundputzer® medium

Removes viscous, firm and/or necrotic coatings, may absorb plenty of detritus.

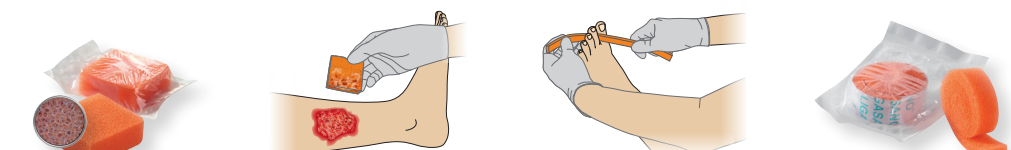


LIGASANO® Wundputzer® interdigital medium

Removes viscous, firm and/or necrotic coatings, may absorb plenty of detritus.

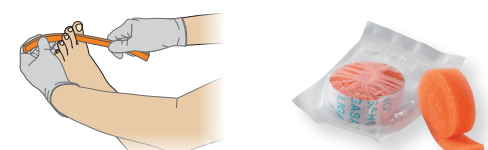


LIGASANO® Wundputzer® soft

Removes soft coatings and biofilm.

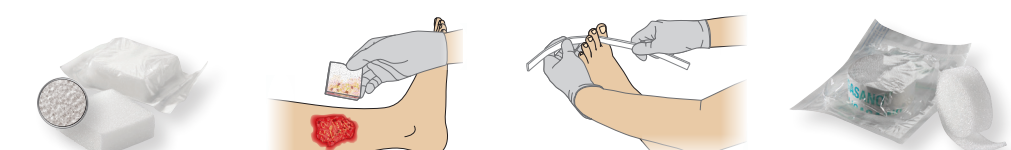

LIGASANO® Wundputzer® interdigital soft

Removes soft coatings and biofilm.

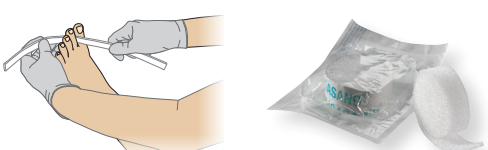

Wound Cleaning / Mechanical Debridement

The **rapidly falling pressure-tension** of LIGASANO® white reduces the pressure on the wound, promotes granulation and additionally reduces the counter-pressure on the new generated granulation.

The **mechanical stimulus** (that means the structure of the dressing is sensed by the tissue) promotes the local blood flow and thus activates self-cleaning.

Because only surplus exudate is absorbed (**controlled suction effect, exudate management**), LIGASANO® white does not dry out the wound. Depending on its thickness, an intermittent under-pressure is generated.


Application Tips !

- always wound contact over the whole surface
- LIGASANO® wound dressing all in all at least 2cm thick
- dressing change as described in the users manual

Fixation

- LIGAMED® fix or parts of it
- Stretch net pants, fixation pants, tubular bandage, etc.
- Air-permeable adhesive plaster


Skin-deep wound (up to a depth of 0.5 cm)

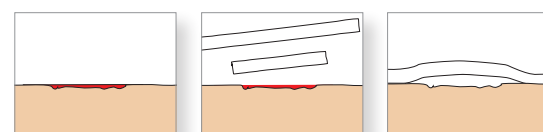


Ulcus cruris venosum




Deep wound

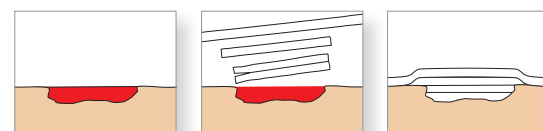


Ulcus cruris arteriosum

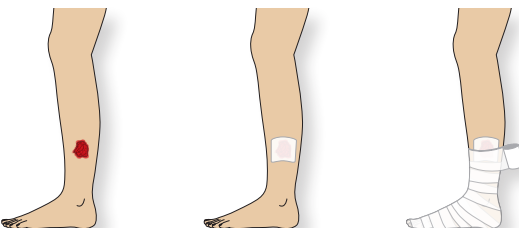


Undermined deep wound

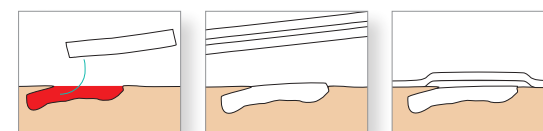

Ulcus cruris mixtum

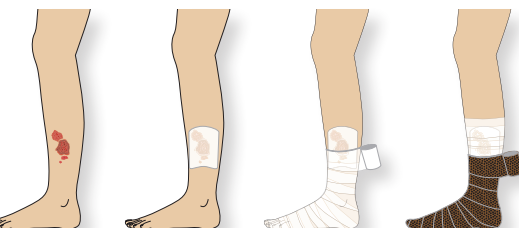

Deep wound with narrow opening / fistulae

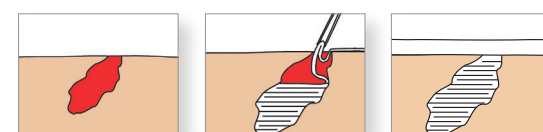


Wound Healing

The **rapidly falling pressure tension** of LIGASANO® white provides an anatomical adaption of the pressure tension to body contours. The dreaded pressure points are avoided by an almost even distribution of support pressure. LIGASANO® white and LIGASANO® green can be simply cut to fit. This is how aids such as pulleys, wedges and heel shoes etc. are created for changing patients' positions, having been adapted by imaginative nursing staff.

In contact to skin **LIGASANO® white** promotes the peripheral perfusion, and increases the patients' pressure resistance. Sweat is absorbed, thus the patient has a normal skin moisture and in combination with the mechanical stimulus additional skin care. The preservation of a physiological skin milieu is supported by LIGASANO® white.

LIGASANO® green is permeable to air and liquid like a sieve and thus avoids accumulation of heat and moisture. The ideal supplement to LIGASANO® white.

Example: individual bed sheet / pressure relief by pressure distribution

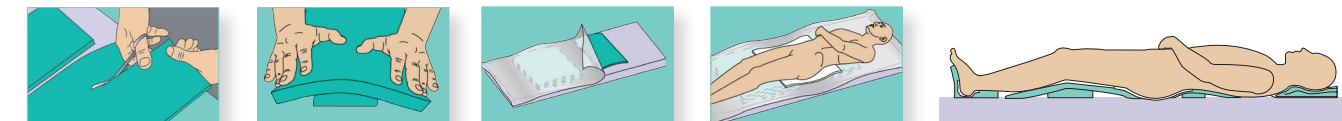

Example: Individual heel shoe




Example: heel cap



Example: prophylaxis and therapy of intertrigo




Example: finger spreader / contracture prophylaxis and therapy

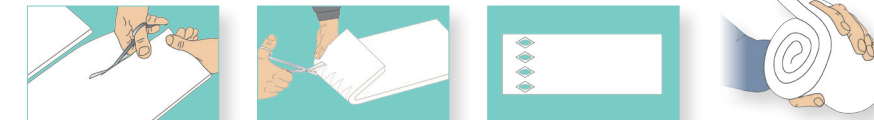

These are only a few examples of the wide application range of LIGASANO® white and LIGASANO® green.

Please find many further examples in our Compendium Wound Cleaning - Wound Healing - Prevention in chapter "Prevention".

Example: finger spreader / individual palm dressing at atonic paralysis

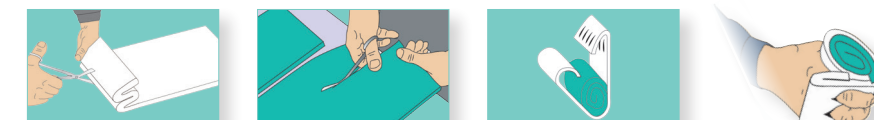

Prevention

You can buy our products at pharmacies or medical supply stores. Reseller can also purchase at wholesale. If an article is not on hand at the wholesaler, you can also order directly from us.

You can reach us by telephone Monday to Thursday from 08.00 to 16.00 and Friday from 08.00 to 12.00. Outside our working hours, please send us an e-mail or tape your request to our answering machine.

We deliver of course also worldwide. We advise you the cost for shipping on request. Because this cost vary to size, weight and country, we can number the total cost only after a precise request or order.

LIGAMED® medical Produkte GmbH

Pfannenstielstr. 2 • D - 90556 Cadolzburg


Tel. +49 9103 / 2046 • Fax +49 9103 / 2796

E-Mail: info@ligamed.de

www.ligasano.com

The copyright for all texts, illustrations and pictures in this brochure owns LIGAMED® medical Produkte GmbH in D-90556 Cadolzburg. LIGASANO®, LIGAMED®, KLIMAGITTER®, MOLTOSAN®, MOLTOMED®, CAVASAN® and Dr. Guck® are registered trademarks of LIGAMED® medical Produkte GmbH.

LIGASANO®
Simply works

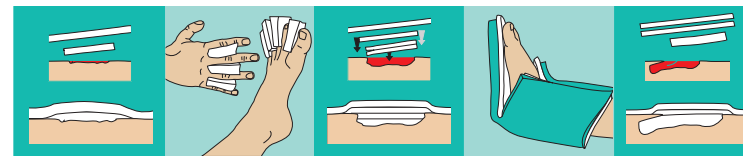

Info 02042 Edition March 2018